


MANUAL MODE


You could put the chair in a manual mode by pulling the handle located on the right cane of the backrest.

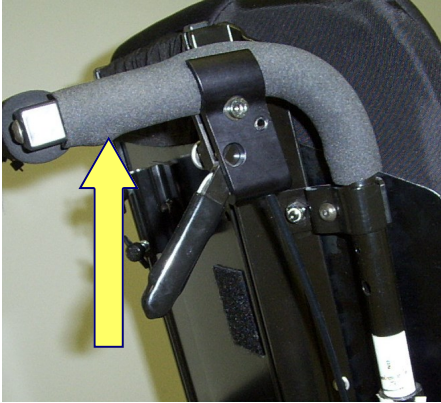

TRANSFERS

Before transferring in or out of the wheelchair, it is important to make sure that these following safety measures are applied.

1. The wheelchair must be **TURNED OFF** before sitting in or standing up from the wheelchair. You must always make sure that all the lights on the control box are **OFF** before transferring in or out of the wheelchair.
2. Before standing up, the safety belt must be **unfastened**.
3. The footrests must be swung to the sides in order to free up space in front of the seat. In order to do this, you must press on the **metallic** button to swing footrests to the sides.

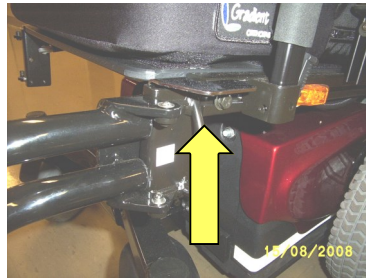

4. You can swing the control box to the side by pushing on the black button. Pull the control box back to its original position. You will feel or hear a click.



Oasis

MOTORIZED WHEELCHAIR



Constance-Lethbridge

Rehabilitation Center

7005, de Maisonneuve O.

Montreal Québec

H4B 1T3




TURNING ON

To operate the wheelchair, press on the green button located at the top of the control box. A series of bars will appear on the screen.

TURNING OFF

To turn off the wheelchair, press the same green button. Lights will go out.

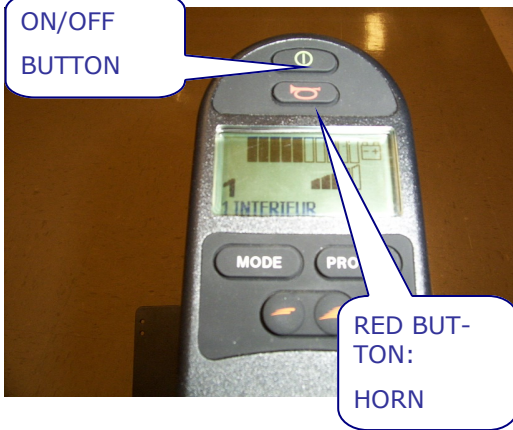

HORN

The button is located just under the on/off button.

DRIVING WITH THE JOY-STICK

Wheelchair will advance in the direction that you point the joystick. The further you push the joystick, the faster the wheelchair will advance.



PROFILE INDICATOR

The OASIS profile are represented by the number 1 to 5 located at the lower left corner of the screen (1=slowest, 5=fastest).

MODE BUTTON

Mode button is only functional if you have electric components.
Ex: seat tilt, motorized leg rest.

CHANGING SPEEDS

The profile button is used to change profiles of the wheelchair. When pressed, the profile indicator will show you in which profile you are in.



Once the desired profile is chosen, you may also turn the variable speed knob located on the lower left side to adjust the speed within that setting.

To **reduce** the speed, turn the knob **counterclockwise**

To **increase** the speed, turn the knob **clockwise**

CAUTION : do not turn the knob while driving!

DISPLAY FOR BATTERY CHARGE LEVEL

The display for the battery charge is located at the top of the screen. The load is represented by a series of bars which lose their color when the battery load decreases. It is recommended to reload the batteries when half of the bars have changed color.

BATTERY CHARGE

To charge the batteries, you must unwind the extension cord behind the backrest. Plug it into a wall outlet.

To ensure your safety and to avoid damage to the wheelchair, the battery must be charged in one extended period rather than short periods at a time.

