

MANUAL MODE

To put the wheelchair in manual mode, you must push down the levers on each side of the motors. Pull them up to return to driving mode.



out of the
int to make
ving safety
measures are applied.

1. The wheelchair must be **TURNED OFF**. You must always make sure that all the lights on the control box are **out** before transferring in or out of the wheelchair.
2. Before standing up, the safety belt must be **unfastened**.
3. The footrests must be swung to either sides in order to free up space in front of the seat. In order to do this, you must press on the **black** lever to swing them away.



4 . You can swing the control box to the side by pushing on the support and do the reverse to bring it back to its original position.



STORM

MOTORIZED WHEELCHAIR



Constance-Lethbridge

Rehabilitation Center

7005, de Maisonneuve O.

Montreal Québec

H4B 1T3

TURNING ON

To operate the wheelchair, pull up the lever located on the left side of the control box. The screen will appear.

TURNING OFF

To turn off the wheelchair, pull down the same lever. The screen will go out.



DRIVING WITH THE JOY-STICK

Wheelchair will advance in the direction that you point the joystick. The further you push the joystick, the faster the wheelchair will advance.



SPEED PROFILE INDICATOR

The STORM five speed profiles are indicated in the top left of the screen by a number following the "D" letter.

A speed level is also adjustable within each program by using the right lever.

CHANGING SPEEDS PROFILES

The on/off lever is also used for changing the speed profile by pushing it up again. The number following the "D" letter will change and telling you in which profile you are in.



DISPLAY FOR BATTERY CHARGE LEVEL

The series of black bars indicating the battery charge is located on the right side of the screen. As the battery load decreases, bars will lose their color.

BATTERY CHARGE

When half of the black bars indicating the battery charge have lost their color, it is time to recharge them.

Connect the round connector in the control box and the electric wire of the charger to the wall outlet.



Charger:
Electric wire for
standard wall
outlet